Medical Splint Instructions

EN [Product Name] Medical Splint

[Modle]AZ-ST, AZ-ST-R100XL, AZ-ST-R100L, AZ-ST-R100, AZ-ST-R92XL, AZ-ST-R92L, AZ-ST-R92, AZ-ST-R76, AZ-ST-R61, AZ-ST-R50, AZ-ST-R46, AZ-ST-R45, AZ-ST-F100XL, AZ-ST-F100L, AZ-ST-F100, AZ-ST-F92XL, AZ-ST-F92L, AZ-ST-F92, AZ-ST-F76, AZ-ST-F61, AZ-ST-F50, AZ-ST-F46, AZ-ST-F45, AZ-ST-25, AZ-ST-23, AZ-ST-12, AZ-ST-05, AZ-ST-04, AZ-WLP, AZ-WLP-01, AZ-WLP-02, AZ-WLP-03, AZ-WLP-04, AZ-WLP-05, AZ-WLP-06, AZ-WLP-07, AZ-WLP-08, AZ-WLP-09, AZ-WST, AZ-WST-01, AZ-WST-02, AZ-WST-03, AZ-WST-04, AZ-FCK, AZ-FCK-01, AZ-FCK-02, AZ-STK, AZ-SAB, AZ-RWA, AZ-RWB, AZ-WBA, AZ-WBB, AZ-KB, AZ-MST, AZ-AST

[Intended Use]

It is physical supports for injured patients to secure or support patients in first aids or physical therapy.

TheMedical Splint is worn on the patients' upper or lower extremities to support, correct and prevent deformities, or align bodystructures for functional improvement.

[Application]

The medical splint is the emergency splint of choice for: First Responders, EMS, Military Medics, Athletic Trainers, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians, and more.

[Directions for use]

1.Make the medical splint to suitable shape to fracture part

2.Fixation together fracture part and medical splint with bandage or straps

[Cleaning and Disinfection]

If you decide to re-use Splint, it should be initially cleaned using acloth or sponge soaked with soap and water or a mild detergent to remove gross contamination. Any blood, body fluids, tissue or other soil materials should be completely removed prior to disinfection.

Next, prepare a dilution of sodium hypochlorite bleach (for example Clorox, White King) for a final concentration of 0.625% to 0.7%. This can typically be done with one-part bleach to 9 parts of water with normal household bleach, or one-partbleach to 11 parts of water for“concentrated” bleach.

Fully immerse Splint in the diluted bleach solution ensuring all air bubbles are removed from the Splint by wiping each of the surfaces (including the sides) with a clean cloth while in the bleach solution. Allow Splint to remain submerged in the bleach solution for 10 minutes following wiping.

ALWAYS rinse Splint thoroughly with running water following immersion then allow to air dry or towel dry before placing it back into service.

[Storage] Avoid high temperature & moisture

[Shelf life] Five years

 **Attention**

Please use under the guidance of the doctor or nurse.

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[Labels, Packing Logo Design]

| Symbol | Introductions             | Symbol | Introductions                                       |
|--------|---------------------------|--------|-----------------------------------------------------|
|        | CE Symbol                 |        | AUTHORISED REPRESENTATIVE IN THE EUROPEAN COMMUNITY |
|        | medical device            |        | AUTHORISED REPRESENTATIVE IN THE SWITZERLAND        |
|        | Manufacturer Name Address |        | Manufacture Date                                    |
|        | Batch Code                |        | Use-by date                                         |
|        | Warnings and Precautions  |        | Consult instructions for use                        |
|        | Catalog Number            |        | Do not use if package is damaged                    |
|        | Unique Device Identifier  |        | Importer                                            |

 Anping Guardian Medical Equipment Co., Ltd.  
500 meters, Nan Sucun, Anping County,  
Hengshui City, Hebei Province,053600,  
China.Tel:+86 13932089888  
www.med-resq.com

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MedNet EC-REP GmbH
MedNet SWISS GmbH