

SUPPORTS  
INSTRUCTION MANUAL

## Medical Splint Instructions

### [Product Name] Medical Splint / Traction Splint

#### [Intended Use]

It is physical supports for injured patients to secure or support patients in first aids or physical therapy. The Medical Splint is worn on the patients' upper or lower extremities to support, correct and prevent deformities, or align body structures for functional improvement.

#### [Application]

The medical splint is the emergency splint of choice for: Military Medics, Athletic Trainers, First Responders, EMS, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians, and more.

#### [Directions for use]

1. Make the medical splint to suitable shape to fracture part  
Fixation together fracture part and medical splint with bandage or straps

Feature:

Lightweight and compact  
quickly immobilize an injured limb  
Waterproof and Radiolucent  
Latex Free

#### [Cleaning and Disinfection]

1. Clean the stretcher with warm, soapy water and a cloth or soft brush, or wipe with a mild solvent after each use, and dry it in air.  
2. Wipe all surfaces with a disinfectant (for example, Accel® PREvention™ Wipes and RTU). Please follow disinfectant manufacturer's instructions.

[Storage] Avoid high temperature & moisture

[Shelf life] Five years



Please use under the guidance of the doctor or nurse



Anping Guardian Medical Equipment Co., Ltd.  
500 meters, Nan Sucun, Anping County, Hengshui City, Hebei Province, 053600, China.



**MedNet EC- REP GmbH**  
Borkstrasse 10, 48163 Münster, Germany



Date: Aug. 31, 2020

Version: A/1