

CVN Arctic Halti Rescue Blanket

Intended use:

Alleviate hypothermia.

Intended users:

Military, EMS, Hospital, Fire Service, Police, Search & Rescue, Veterinarians



ENG: Halti Rescue blanket instructions for use:

1. Unpack the product and spread it on the ground with the silver side up.
2. Lay the patient on one side of the rescue blanket.
3. Remove the protective paper from the adhesive label on the edge of the blanket.
4. Fold the other edge over the patient so that only the face is visible and attach the edge on the tape.

SWE: Halti rescue blanket instruktioner:

1. Packa upp täcket och sprid ut den på marken med silver-sidan uppåt.
2. Lägg patienten på ena sidan av täcket.
3. Ta bort skyddspapperet från den självhäftande tejpens på täckets kant.
4. Vik den andra kanten över patienten så att endast ansiktet syns och klistra fast den på tejpens.

DAN: Halti redningstæppe brugsanvisning:

1. Pak produktet ud og spred det på jorden med sølvsiden opad.
2. Læg patienten på den ene side af redningstæppet.
3. Fjern beskyttelsespapiret fra den selvklæbende tape på kanten af tæppet.
4. Fold den anden kant over patienten, så kun ansigtet er synligt, og fastgør kanten på tapen.

NOR: Halti Rescue teppe bruksanvisning:

1. Pakk ut produktet og legg det på bakken med sølvsiden opp.
2. Legg pasienten på den ene siden av redningsteppet.
3. Fjern beskyttelsespapiret fra selvklebende etikett på kanten av teppet.
4. Brett den andre kanten over pasienten slik at bare ansiktet er synlig og fest kanten på tapen.

FIN: Halti Rescue -peiton käyttöohjeet:

1. Pura tuote pakkauksesta ja levitä se maahan hopea puoli ylöspäin.
2. Aseta potilas pelastuspeiton toiselle reunalle.
3. Poista suojapaperi peiton reunalla olevasta tarrasta.
4. Taita toinen reuna potilaan yli niin, että vain kasvot ovat näkyvissä ja kiinnitä reuna peittoon teipillä.



Store in a cool dry place, out of direct sunlight, off the floor.



n/a

Adverse event notice:

Carevision Oy
Hiltusentie 26 A 2
90620 Oulu
FINAND
+358456431439
info@cvn.fi



Orvec International Limited
 Malmo Road
 Hull
 HU7 0YF



CS Lifesciences Europe Ltd, 3 Inn Quay,
 Dublin 7, Ireland, eurep@cslifesciences.com



Orvec International Limited
 Malmo Road
 Hull
 HU7 0YF

CAREVISION OY NCAGE: A843G, NSN: 6532-01-524-6932



3004493629



MEDICAL SOLUTIONS
 FOR DEMANDING SITUATIONS