Nomex Model

- NL** Handleiding & Veiligheidsinformatie
- EN** User Manual & Safety Information
- DE** Gebrauchsanleitung & Sicherheitshinweise
- NO** Bruksanvisning og sikkerhetsinformasjon
- DA** Brugervejledning & Sikkerhedsinformation
- SV** Användarhandbok
- FI** Käyttöohjeet ja turvallisuustiedot

Retmex Reddingsdoek - Nomex Model

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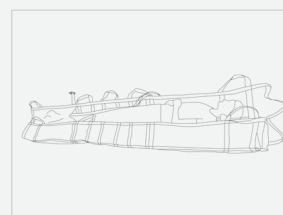
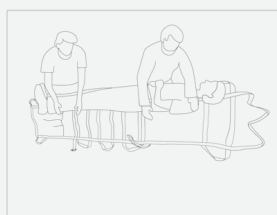
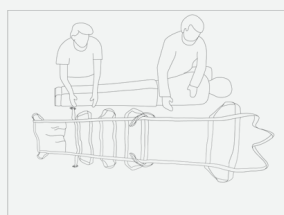
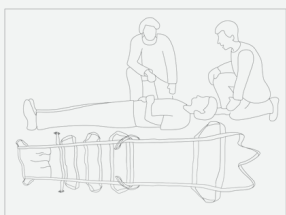
Retmex Rescue Sheet - Nomex Model

This manual will help you to use the Retmex Rescue Sheet - Nomex Model in a correct and safe way.

Please refer to these instructions before using the product.

The Retmex is a 2-in-1 rescue solution. It has two functions:

- Saving a victim in the most critical circumstances (house fires, riots, terrorist attacks, etc.)
- Fire extinguishing (e.g. when a person's clothing is on fire)

This makes the Retmex one of a kind. To overcome the most critical situations, the rescue cloth is made of first-class materials:

Nomex®, Kevlar®, fire retardant retro-reflective tape and an advanced fire-retardant tape construction.

Advice for training

Practice using the Retmex in order to guarantee its correct and safe use in the event of an emergency.

Storage advice

After use, always check the product for damage. Note that if the casualty has already been dragged for a significant distance the structure of the fabric may have been damaged. The Retmex can be cleaned in the washing machine. Follow the washing instructions.

Washing instruction

60 degrees, dry on low temperature, do not bleach, iron on low temperature

Safety information

- Always observe the instruction given in the instruction manual when using the Retmex
- The Retmex is not intended for recreational use.
- Only use the Retmex in emergencies or during training.
- The use of alcohol and/or prescription and over-the-counter medicine may have a negative effect on the skill of the person operating the Retmex.
- Sharp edges or protrusions may damage the Retmex and, therefore, reduce the effectiveness of the product.
- Not observing the instructions given in the instruction manual may result in serious physical injury.
- The manufacturer is not liable for (consequential) damage as a result of the use of the Retmex.
- Extinguishing a fire can be dangerous. If this is not done correctly, it can lead to injuries.

Replacing the Retmex Rescue Sheet

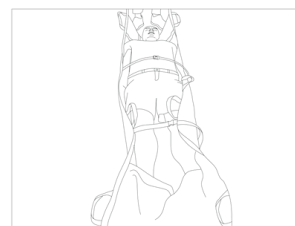
- Replace the Retmex if it is discolored by heat.
- Replace the Retmex if it is damaged by heat or friction.
- Replace the Retmex if stitches on the Retmex are severely damaged by heat or friction.

Preparing for use (A)

1. Release the hook and loop fasteners and remove the Retmex from its storage system.
2. Roll the Retmex open. Make sure the black webbing is faced upwards. This is the lying area.
3. Open the two clips and place them next to the Retmex (not on the lying area).

Placing a victim onto the Retmex (B)

1. Place the victim in a lateral position and place the Retmex next to the victim (in the back).
2. Now roll the victim from the lateral position onto the Retmex.
3. Place the victim in the centre of the Retmex.
4. Place the feet in the feet section.
5. Close the clips around the torso and legs. Tighten the clips.
6. Use the pulling straps to transport the victim.

Horizontal evacuation

It is recommended that you evacuate the victim with 2 persons.

It is possible to do this on your own, but with 2 persons the psychological load will be reduced.

1. Use the strap or handles at the head or foot end to move the person. If you use the pulling strap at the head end, make sure to lift the victim's head and neck for support.
2. Use your weight to move the victim on the Retmex. Take a stand like a 'water skier'. Keep your arms straight.

Staircase evacuation

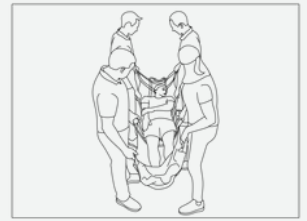
It is recommended to do the vertical evacuation with at least 2 persons.

1. Make sure there is one person at the foot end and one person at the head end.
2. The person's feet on the rescue sheet should be in the sliding direction.
3. The person at the foot end will step down a few steps and pull the rescue sheet towards him. Use the handles for doing so.
4. If the person's buttocks are over the edge of the first step, there will be a tipping point. The person at the head end then uses the long strap at the head end to lift the rescue sheet. This creates a wave motion. Make sure to lift up the strap at the head end to provide support to the victim's head and neck.

Attention! The fabric is thin, which means that the person on the Retmex might hit the stairs with his buttocks. This can be sensitive. Do not overestimate your skills, strengths and abilities. Staircase evacuation can be difficult. You must be able to control the victim's descent speed while sliding down the staircases.

Transfer a victim to the stretcher

Use the side handles on the longitudinal sides to transfer a person to a stretcher. With 3-6 people you can easily and quickly find balance which makes the transfer to the stretcher run smoothly. Communicate with your colleagues before you start the transfer.

Extinguish a fire with the Retmex

Due to Nomex' heat resistance and solid structure, the rescue sheet has special capabilities: it can extinguish fire. Attention! The rescue sheet is not an official fire blanket and is therefore not an alternative. There are legal requirements for fire blankets with regard to dimensions and design. Extinguishing a fire can be dangerous. If this is not done correctly, it can lead to (serious) injuries.

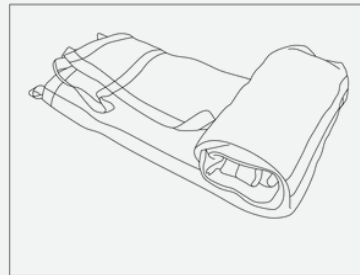
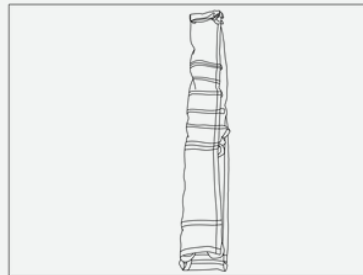
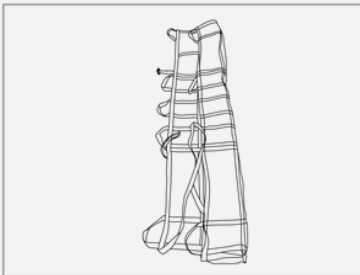
Extinguishing fire using this step-by-step instruction:

- Protect your hands. Wear fire resistant gloves or wrap the top of the Retmex around your hands to protect them.
- Keep the Retmex high and stay behind the Retmex.
- Place the Retmex over the fire. Do not throw, but lay it gently. Make sure there is no air left. Leave the Retmex in place for a while.
- Wrap the victim from top to bottom. This will prevent the flames from reaching the face. Do not cover the victim's head when extinguishing the fire.
- Rub from top to bottom until the fire suffocates.

Storing the Retmex

1. Open the rescue sheet and place the section on the right-side inwards.
2. Turn the left part in.
3. Roll up the Retmex and place it back in the yellow storage system.

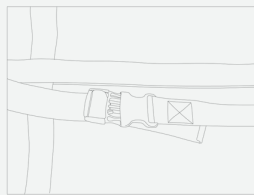
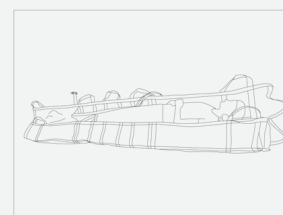
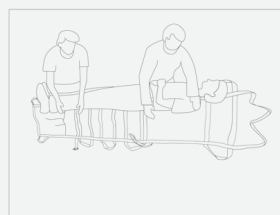
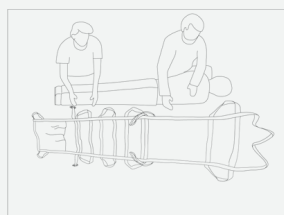
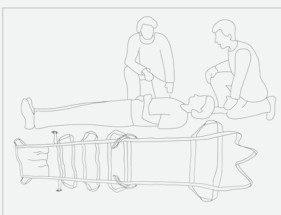
Please note that all handles and straps are placed inwards when storing the Retmex.

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