

# CVN TQ

## TECHNICAL SPECIFICATIONS

### PACKAGE DIMENSIONS AND WEIGHT

Width: 7cm  
Length: 14,5 cm  
Height: 5 cm  
Weight: 95 g

### APPLICATION DIMENSIONS

Length: 95 cm




● H5000-4013T



**CVN TQ** Extremity tourniquet should only be used in health care applications and for use by health care personnel.

## REMOVAL INSTRUCTIONS


1. STEP

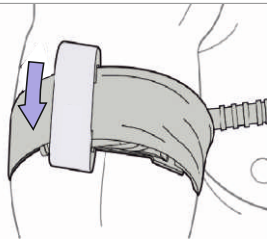

Do not touch the tourniquet until you are directed to any health care personnel.

Release the time velcro and the remove the W-rod from the fastening section.

2. STEP

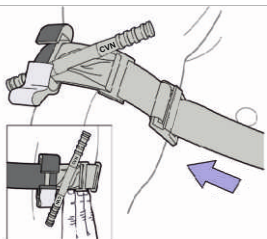

Remove the tourniquet from the limb of the injured person by releasing the band you fixed from the Velcro.

04/2023 1404		LOT 20230401		EXTREMITY TOURNIQUET	
CAREVISION OY NCAGE: A843G		BASIC UDI-DI: 8682169339324C2		CVN TQ	
04/2023		For a Gapp Savunma Tıbbi Oğ. Dan. A.İ.İ. Ltd. Şti. Sarı Oğ. Dan. A.İ.İ. Men. 1404. Sk. No. 4 Etilmesquı/ANKARATÜRKİYE SRN: TR-MR000034360		CE	
CAREVISION OY NCAGE: A843G		BASIC UDI-DI: 8682169339324C2		REF	
04/2023		Made For: CVN Medical Solutions Atene Şişli Hastane 26.42 Fl.06020 Culu, Finland SRN: FI-M-000016330		i	




After purchasing the product, take it out from its packaging and try it.



# CVN TQ EXTREMITY TOURNIQUET


## INSTRUCTIONS FOR USE

### 1. STEP

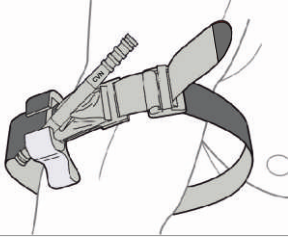


Locate the injured area, place the CVN TQ around the limb onto 5-6 cm (2-3 in) above the wound.

### 2. STEP

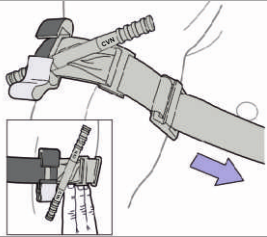


Pull the band (Strap) tight and fasten it around the limb with velcro again back on velcro. The band should be tight so that the tips of the 3 fingers should not slip between the band and the limb. If the tips of three fingers slip under the band,

### 3. STEP

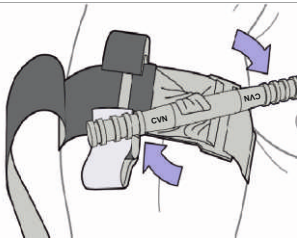


Twist the W-rod, until the bleeding stops.

### 4. STEP

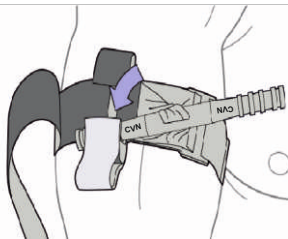

After the bleeding stops, secure the W-rod into the C-Lock. Check for bleeding and distal pulse. If bleeding is uncontrollable or there is a distal pulse, apply an additional tourniquet with a second CVN TQ above and side-by-side to the first CVN TQ.

### 5. STEP

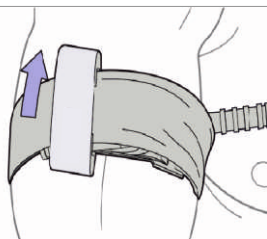

Wrap the band through the C-Lock and over the W-rod.

Fix the time velcro on the band and W-rod and record the application time.



CVN TQ should not be applied over hard objects on clothing.

If the injury site is not easily visible, place the CVN TQ on the arm or leg as high and firmly as possible. When intervention is more appropriate, reconsider the injured limb and place the CVN TQ directly on the skin 5-6 cm (2-3 in) above the wound.