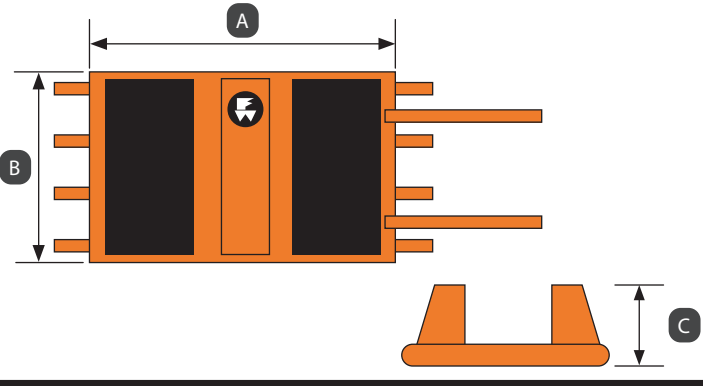
Model 445 Universal Head Immobilizer

Model 445

A	16" (410 mm)
B	10" (260 mm)
C	6" (160 mm)
	2 lb (0.86 kg)

Ferno-Washington, Inc.
70 Weil Way
Wilmington, OH 45177-9371, U.S.A.
1.937.382.1451

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Ferno (UK) Ltd.
Ferno House, Stubs Beck Lane, Cleckheaton,
West Yorkshire, BD19 4TZ, UK
+44 (0) 1274 851 999

Description

Ferno® Model 445 Universal Head Immobilizer is an emergency patient-handling device designed to aid in immobilizing a patient's head and neck. It is for professional use only.

The head immobilizer is vinyl coated, waterproof and buoyant. It is designed not to interfere with medical scanners, and may be left in place during X-rays, MRI or CT scans.

General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. A minimum of two trained operators is required. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the head immobilizer following standard emergency patient-handling procedures. Keep training records.
- Inspect the head immobilizer and confirm it is in good condition.
- Attaching: Thread side straps through backboard or scoop stretcher slots and fasten to base plate. Attach top strap through center hole (if present). Adjust and tighten straps; confirm base plate does not slip on device.
- Follow standard emergency patient-handling procedures when operating the head immobilizer. Qualified personnel should assess the patient's condition and determine the proper procedures and equipment to use.
- Typical operation is for one operator to support the patient's head and neck while the second operator applies the head immobilizer. Center the patient's head on the base plate. Position the head pads against the patient's shoulders if possible.
- Use a cervical extrication collar with head immobilizer when possible.
- Secure patient on backboard or scoop stretcher using approved patient restraints.
- Stay with the patient at all times.

Disinfecting and Cleaning

Remove from backboard or scoop stretcher before cleaning and disinfecting. Replace if ripped, punctured or damaged.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information www.ferno.com
 Authorized Service ferno.com/service-and-support

U.S., Canada	1.877.733.0911
International	Country Code +1.937.382.1451
Fax	1.888.388.1349

Complete English Users' Manual available at: www.ferno.com

PATENTS
ferno.com/patents

WARNING

Professional use by trained operators only. Use the product only for its intended purpose.

Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.

An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.

Improper parts, modifications, or items not designed for the product can cause injury. Use only Ferno-approved parts and service. Attach only approved items. Do not modify the product.

Symbol	Definition	Symbol	Definition
	Follow training protocol		Cleaning
	Disinfecting		Product weight

NOTICE

Affix base plate firmly to backboard for proper function. If it slips, use a different backboard.

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the product.

Inspection Checklist

- Are all components present?
- Is head immobilizer free of excessive wear, punctures, frayed edges or tears?
- Does base plate remain firmly in place (does not slip) when secured to the backboard or scoop stretcher?

If inspection shows damage or excessive wear, remove the head immobilizer from service until repair is made.

Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at tscoordinator@ferno.com. If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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Information www.ferno.com
 Autoriseret service ferno.com/service-and-support

USA, Canada	1.877.733.0911
International	Landekode +1.937.382.1451
Fax	1.888.388.1349

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