

Description

The Ferno CPR Board provides a firm surface for CPR training and administration. The head cup maintains proper alignment of patient's head and helps to properly position the patient and ensure a constant, open airway. Contoured sides make lifting the patient easier.

Usage

The CPR Board should be slid under the patient, ensuring that the patient's head is situated in the rounded head cup in the board to maintain proper alignment to perform CPR.

Cleaning & Disinfection

The CPR Board is made of ridged, high resistant polyethylene. To clean, wipe with cleaning and disinfecting solution.

Specifications

Length: 60 cm
Width: 42 cm
Depth (thickest point): 6.35 cm



CPR Board in use

DISCLAIMER

This manual contains general instructions for the use, operation and care of this product. The instructions are not all-inclusive. Safe and proper use of this product is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations and local protocol. Training on the proper use of this product must be provided before using this product in an actual situation.

Retain this manual for future reference. Include it with the product in the event of transfer to new users. Additional copies are available upon request through Customer Service.

PROPRIETARY NOTICE

The information disclosed in this manual is the property of Ferno Canada, Mississauga, Ontario, Canada.

Ferno Canada reserves all intellectual property rights, proprietary design rights, manufacturing rights, reproduction use rights, and sales use rights thereto, and to any article disclosed therein except to the extent those rights are expressly granted to others or where not applicable to vendor proprietary parts.



MEDICAL DEVICE
GTN/UDI 00754016222078



Made in China for Ferno Canada
2460 Tedlo Street, Mississauga
Ontario L5A 3V3, Canada
+1-866-998-7866

EC

REP

FERNO S.r.l.
via B. Zallone, n. 26, 40066
Pieve di Cento, Bologna, Italy
+39.051.6860028

1. Symbol Glossary

The symbols defined here are used on the CPR board and/or in this manual. Ferno uses symbols recognized by the International Standards Organization (ISO), American National Standards Institute (ANSI) and the emergency medical services industry.



Reference Number
(Product Name)



Batch Number



Date of Manufacture



Read the Users' Manual



Manufacturer



Authorized Representative in the European
Community



Product Meets European Union Standards



Load Capacity



General Warning of Potential Injury

NOTICE

Training is required before using this CPR Board. Refer to the provided instructions before use. Check CPR Board for wear and tear before every use and retire if any visible signs of damage are found.

2. Operator Skills And Training

2.1 Skills

Operators using the CPR Board need:

1. A working knowledge of emergency patient-handling procedures.
2. Follow local protocols and guidelines as set down by your organization.

2.2 Training

Trainees need to:

1. Follow a training program designed or approved by their training officer.
2. Read this manual. For additional free users' manuals, contact Ferno Canada Customer Service.



WARNING

Untrained operators can cause injury or be injured. Permit only trained personnel to use the CPR board.

CUSTOMER SERVICE

Customer relations and product support are important aspects of each Ferno Canada product. For assistance with the stretcher please contact your local distributor or Ferno Canada Customer Service as listed below:

FERNO CANADA

2460 Tedlo Street, Mississauga, ON, L5A 3V3, Canada

Telephone (Canada)

1.800.543.3766

Website

www.ferno.ca

EUROPEAN REPRESENTATIVE

Ferno S.r.l. via B. Zallone, n.26, 40066 Pieve di Cento, Bologna, Italy

Telephone (Europe)

39.051.6860028