

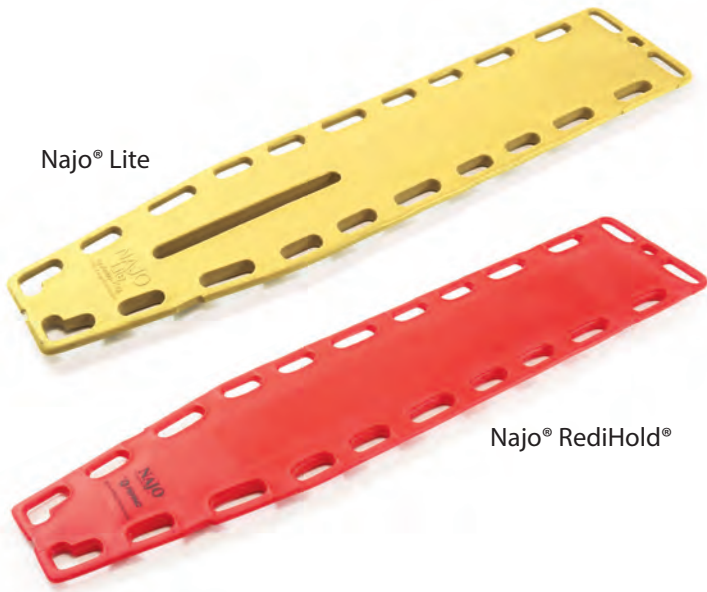
## Najo® Backboard Series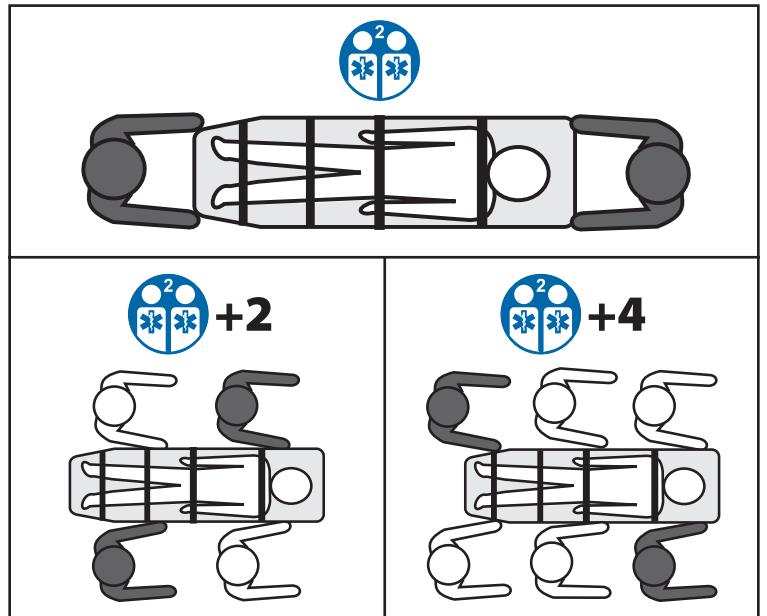

	Najo Lite	RediHold®
<b>A</b>	72" (1830 mm)	72" (1830 mm)
<b>B</b>	16" (410 mm)	16" (410 mm)
<b>C</b>	1.75" (45 mm)	1.75" (45 mm)
	14.5 lb (6.6 kg)	16 lb (7.2 kg)
	450 lb (204 kg)	600 lb (272 kg)
	154 lb (69 kg)	136 lb (62 kg)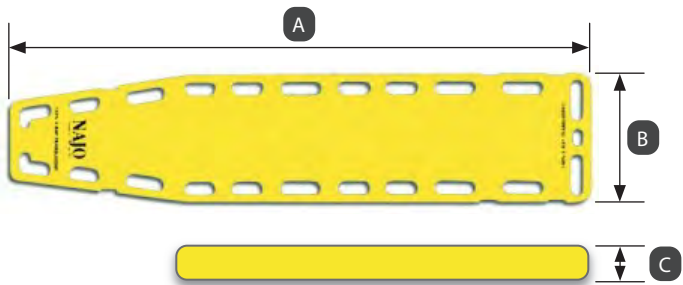

Ferno-Washington, Inc.  
70 Weil Way  
Wilmington, OH 45177-9371, U.S.A.  
1.937.382.1451

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Ferno (UK) Ltd.  
Ferno House, Stubs Beck Lane, Cleckheaton,  
West Yorkshire, BD19 4TZ, UK  
+44 (0) 1274 851 999

## Description

Ferno® Najo® backboards are emergency hand-carried stretchers designed to provide rigid support for a patient with suspected spine, hip, pelvis, or lower extremity injury. It is for professional use only.

## General Guidelines for Use

Medical advice is beyond the parameters of this manual. It is the operators' responsibility to ensure safe practices during use of the product. A minimum of two trained operators is required. Operators must:

- Read and understand this manual.
- Train and practice on proper use of the backboard following standard emergency patient-handling procedures. Keep training records.
- Inspect the backboard and confirm it is in good condition.
- Communicate with one another and use coordinated movements to maintain control of the backboard.
- Stay with the patient at all times.
- Use additional trained help when working with heavy loads (patient plus equipment).
- Always use patient restraints, following the instructions in the restraint users' manual. Follow local service's protocols and government requirements.
- If speed-clip pins are installed, confirm the restraint speed clips are securely attached to the pins.
- Do not attach speed-clip pins or use the handholds as part of a technical rescue lifting system. Lift and carry the backboard only by hand, grasping the handholds. Place the backboard inside a basket for technical rescue.

## Disinfecting and Cleaning

Remove the restraints from the backboard before cleaning and disinfecting. Refer to the restraints users' manual for instructions.

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time.

Hand clean with warm water and a mild detergent. Rinse with warm, clear water. Dry with a towel or allow it to air-dry.

## Ferno Customer Relations

For ordering assistance or general information, contact your Ferno distributor or Ferno Customer Relations.

Information	<a href="http://www.ferno.com">www.ferno.com</a>
Authorized Service	<a href="http://ferno.com/service-and-support">ferno.com/service-and-support</a>
U.S., Canada	1.877.733.0911
International	+1.937.382.1451
Fax	1.888.388.1349

Complete English Users' Manual available at: [www.ferno.com](http://www.ferno.com).

## WARNING

**Professional use by trained operators only. Use the product only for its intended purpose.**

**Improper use, operation, or maintenance can cause injury. Use, operate, and maintain only as instructed.**

**An unattended or unrestrained patient can be injured. Always stay with the patient and use patient restraints.**

**Improper parts, modifications, or items not designed for the product can cause injury. Use only Ferno-approved parts and service. Attach only approved items. Do not modify the product.**

Symbol	Definition	Symbol	Definition
	Follow training protocol		Load capacity
	Disinfecting		Product weight
	Cleaning		Buoyancy
	Use proper restraints		

## NOTICE

Do not use disinfectants or cleaners containing phenolics or iodines. If bleach is used, rinse with plenty of clear water to remove bleach residue.

Do not use abrasive materials to clean the backboard.

Do not immerse restraint metal buckles, slide bars, and clips in liquids or by washing-machine action.

## Inspection Checklist

- Is the backboard free of cracks, crevices, and punctures?
- If the backboard is fitted with speed clip pins, are all pins solidly installed and free of cracks or damage?
- Is patient-restraint webbing in good condition with no cuts or frayed edges?
- Are patient-restraint buckles free of visible damage and do they operate properly?

If inspection shows damage or excessive wear other than for speed clip pins, remove the backboard from service and destroy to prevent accidental use. Speed clip pins are replaceable (Part # 090-5802).

## Adverse Event Notice

In the event of an adverse event or serious incident related to the use of this device, the end user/operator must report the incident to Ferno at 1-877-733-0911 or via email at [tscoordinator@ferno.com](mailto:tscoordinator@ferno.com). If the incident occurred outside of the U.S., report it to Ferno's Authorized Representative for that area.

## Disposal

Prior to disposal, clean and disinfect the product according to instructions in this manual. Recycle or dispose of the product following local protocols and regulations. Do not dispose of the product in municipal waste.

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Information	<a href="http://www.ferno.com">www.ferno.com</a>
Autoriseret service	<a href="http://ferno.com/service-and-support">ferno.com/service-and-support</a>
USA, Canada	1.877.733.0911
International	+ 1.937.382.1451
Fax	1.888.388.1349

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**PATENTS**  
[ferno.com/patents](http://ferno.com/patents)

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