## DIRECTIONS FOR USE

### Motion Restriction Pillow (MRP) Vacuum Pillow

**English:** Directions for use. **Ferno Motion Restriction Pillow**  
**Svenska:** Bruksanvisning. **Ferno Motion Restriction Pillow**  
**Dansk:** Brugsanvisning. **Ferno Motion Restriction Pillow**  
**Norsk:** Brukerveiledning. **Ferno Motion Restriction Pillow**  
**Suomi:** Käyttöohje. **Germa Alipainetyyny-**

**Distributor:**

Ferno Norden AS  
Innlaget 298  
NO-3185 Skoppum  
Norway

Phone: +47 33 03 45 00

Fax: +47 33 03 45 10

E-mail: [fno@fernonorden.com](mailto:fno@fernonorden.com)

[www.fernonorden.com](http://www.fernonorden.com) / [www.fernonorden.se](http://www.fernonorden.se) / [www.fernonorden.no](http://www.fernonorden.no)

**Manufacturer:**

AB Germa  
Industrigatan 54-56  
SE-291 36 Kristianstad  
Sweden

Phone: +46 (0)44 12 30 30

Fax: +46 (0)44 10 31 79

E-mail: [info@germa.se](mailto:info@germa.se)

[www.germa.se](http://www.germa.se)

English:	Directions for use	Page	3
Svenska:	Bruksanvisning	Sida	6
Dansk:	Brugsanvisning	Side	9
Norsk:	Brukerveiledning	Side	12
Suomi:	Käyttöohje	sive	15

English: These directions for use may be updated without further notice.  
Copies of the current version are available from the manufacturer.

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#### 1. Warning and caution statements

##### CAUTION

- Check regularly that the vacuum pump works correctly and that the air can be evacuated from the MRP Vacuum Pillow so as to make it rigid.
- Avoid handling the close to sharp objects.
- Avoid handling the MRP Vacuum Pillow close to an open fire.
- Check regularly that the vacuum pump, the MRP Vacuum Pillow and the straps are not damaged or display signs of wear and tear. Repair any damages and change defective parts immediately.
- Check regularly that the MRP Vacuum Pillow has no cracks.
- Own modifications on products not allowed – Your medical specialist to be responsible for any modification.
- When decided to take the product out of service and destroy – make sure to follow your local instructions or contact us for additional information.

Recommendation: Usable life of 1 year

#### 2. Intended use

MRP Vacuum Pillow is intended to stabilise patients head during transport. They are suited to support of patients head. MRP Vacuum Pillow is available for head support for most common ambulance stretchers.

#### 3. Specifications

MRP Vacuum Pillow are in conformity with EU MDR 2017/745 Class 1 concerning Medical Device

**Temperature range:** -30 °C to +70 °C

##### Parts/Materials

Cover material:	PVC coated polyester fabric (flame retardant)
Inner bag:	Polyester chamber system
Granules:	Polystyrene granules (flame retardant)
Padding:	Pillow fibers polyester

Dimensions	Thickness (cm)	Width (cm)	Height (cm)	Weight (kg)	Capacity for hand pump
<b>Ferno Motion Restriction Pillow MRP, vacuum</b>	10	48	36	0,8	-450hPa at c. 25 strokes

#### 4. Description

MRP Vacuum Pillow consists of an outer cover made of hard-wearing PVC coated fabric with an innerbag of polyester filled with polystyrene granule and polyester pillow stopping. The inner bag is sewn in channels enabling the granule and pillow stopping to spread evenly.

When the air is evacuated, the granule and pillow stopping is moulded into a tight substance making the Vacuum Pillow rigid, unable to re-shape. Before and while the air is evacuated the MRP Vacuum Pillow may be moulded according to the shape of the patient's head in order to obtain optimum support of the patient.

MRP Vacuum Pillow is equipped with straps with push buttons to assembly the pillow when necessary on the stretcher upper back frame. The valve shall be on the back of the pillow when assembled.

## **5. Operating instructions**

1. Assembly MRP Vacuum Pillow on stretcher back on top of the mattress, in medical correct high for the patient
2. Close the valve by turning it to the right (clockwise).
3. Place the patient on the stretcher medically desired position, place patient head in center of the pillow.
4. Mould the MRP Vacuum Pillow so that it is even or, if necessary, in an appropriate state for the emergency.
5. Mould the MRP Vacuum Pillow so that the patient head is comfortable supported for the patient. For best support, allow the pillow to rest on the shoulders when applying.
6. Connect the pump. Evacuate the air, and continue to mould the pillow.
7. Make sure that the patient is in medial preferred position and comfortable supported.
8. Disconnect the pump.
9. The patient is ready to be transported.
10. Make sure the patient is medically correct and comfortable positioned during transport.

## **6. Cleaning and disinfecting**

After each use the Vacuum Pillow should be wiped with a moist cloth and dried. If dirty the pillow may be washed with soapy water and rinsed with clean water. Be sure to keep the valve closed if rinsed with water.

Allow the MRP Vacuum Pillow to dry completely before storing it.

If the MRP pillow needs disinfection first clean it carefully and let it dry. Wipe the complete splint with a solution containing 70% alcohol. Note: most common disinfectant can be used for clearing.

Test MRP Vacuum Pillow at regular intervals to make sure that it has the necessary tightness.

If leaks are found; see section 7 for repair instructions.

## **7. Maintenance and repair**

At a regular basis MRP Vacuum Pillow and the pump must be checked for visible damages.

At the same time the pillow must be tested as to tightness. Furthermore the suction force of the vacuum pump must be checked according to Section 8.

Leaks can be found on Vacuum Pillow by visual inspection or by filling the pillow with air covering it with soapy water all over the surface and the welds and watching out for air bubbles from the pillow.

If leaks have been found on the pillow, mark the leak, let any air out, clean the area around the leakage and let it dry.

Find a patch in a suitable size and put glue on the patch as well as on the leakage on the pillow. Wait for the glue to become dust dry (approx. 2 minutes) and place the patch on the leakage. Firmly press the whole area of the patch establishing good contact and press out all air under the patch.

The repair will have full strength after 24 hours.

### **8. Test of the Pillow and Pump**

Straighten out the Pillow, **close the valve** by turning it clockwise and connect the pump.

Evacuate the air until the Vacuum Pillow feels rigid and leave it for about 2 hours; if the pillow is as rigid as it was when you started it may be considered tight. Open the valve by turning it counter-clockwise.

If the Vacuum Pillow is not airtight, see section 7 for repair or check the valve for leakage. Try to insert a new valve and repeat the test.

If the pump is not emptying the Vacuum Pillow, check the pump by closing the connection with your fingertip and pump a few times.

A negative pressure should be created in the tubing that can be felt at your fingertip. If the pump does not function do not repair it but return it for service to your dealer or to an Ferno service centre.

### **9. Storage**

Loosen the valve by turning it counter-clockwise and put the Vacuum Pillow on an even surface.

When the granules have been spread evenly, in its initial position.

Repack the MRP Vacuum Pillow so that it fits into your storage area. The Vacuum Pillow is often positioned in original shape in a cover.

### **10. Ordering information – Motion Restriction Pillow**

15 000 000 114 Motion Restriction Pillow (MRP), vacuum

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Claim Report

---

Report no. :  
(To be assigned in by  
Manufacturer)

---

Please print or type or fill in by MS Word, all sections a. – i. Fax or e-mail to your supplier. Please use repair order when reporting repairs not associated with complaints.

---

---

For claims please specify :  
Purchase Date

---

---

Description of product

---

---

Description of Incident/Problem: 1) What, 2) How?

---

---

When? In connection with: 1) transport, 2) unpacking,3) preparing, 4) test, 5) cleaning, 6) normal use, 7) training, 8) disassembly, 9) assembly,10) storage, 11) other:

---

---

Info on the product, prior to problem,

---

Storage: 1) stock, 2) site of use, 3) packed, 4) unpacked, 5) softpack, 6) other:

---

---

Used cleaning and disinfecting agents/Sterilizing method:

---

---

Pattern of use: 1) stand-by for emergency, 2) routine use, 3) stand by, 4) other:

---

Environment: 1) Indoor, 2) Outdoor, 3) Heat, 4) Cold, 5) Humidity, 6) Sunlight, 7) Gases, 8) other:

---

Which actions have been: a) taken, b) planned by the reporter or user at the site of incident? examination of the product on site, 2) replacement of the product, 3) other:

---

Suggested actions by the distributor/complainant

---

---

Returning of samples/faulty product(s),

- 1) Already shipped/Will be shipped to \_\_\_\_\_,
- 2) Pending request from manufacturer,
- 3) Product/parts not available:

---

Date:

---

We acknowledge the receipt of this report by manufacturer and will start processing it immediately. Please use the assigned complaint report number for further reference.

Your contact person in this matter is: \_\_\_\_\_

Name:

Direct Tel.

Direct Fax.

Init/Date:

---