

**S-2108 AF and S-2108 A
POLE STRETCHERS
MANUAL**





Read this manual and retain for future reference

TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum.

Disclaimer

This manual contains general instructions for the use, operation and care of this stretcher. The instructions are not all-inclusive. Safe and proper use of this stretcher is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this stretcher be provided before using this stretcher in an actual situation. Retain this manual for future reference. Include it with the stretcher in the event of transfer to new users. Additional free copies are available upon request from FERNO Manufacturing.

Model S-2108-AF Product Specifications

Length - open206 cm
229 cm
Length - folded103 cm
114 cm
Width - open 55 cm
folded 11 cm
Height - open 14 cm
folded 20 cm
Unit weight 6 Kg
Load capacity159 Kg

WARNING

Untrained operators can cause injury or be injured. Permit only trained personnel to operate the stretcher.

Improper use of the stretcher can cause injury. Use the stretcher only for the purpose described in this manual.

Improper operation can cause injury. Operate the stretcher only as described in this manual.

An unattended patient can be injured. Stay with the patient at all times.

An unrestrained patient can fall off the stretcher and be injured. Use restraints to secure the patient on the stretcher.

An improperly-secured stretcher can separate from the transporter and cause injury. Attach the stretcher securely and verify that the slam latches have locked.

Helpers can cause injury. Maintain control of the stretcher, operate the controls and direct all helpers.

Helpers can be injured. Show helpers where to grasp the stretcher to avoid pinch points.

Improper maintenance can cause injury. Maintain the stretcher only as described in this manual.

Improper parts and service can cause injury. Use only Ferno parts and Ferno - approved service on the stretcher.

Modifying the stretcher can cause injury and damage. Use the stretcher only as designed by Ferno.



Model S-2108-A Product Specifications

Length206 cm
229 cm
Width - open 55 cm
folded 11 cm
Height 14 cm
Unit weight 5 Kg
Load capacity159 Kg

Product Number	Product Name	GTIN/UDI
60-0102-005	S-2108-AF-206 POLE STRETCHER	8588008129579
60-0102-006	S-2108-AF-229 POLE STRETCHER	8588008129579
60-0102-007	S-2108-A-206 POLE STRETCHER	8588008129579
60-0102-008	S-2108-A-229 POLE STRETCHER	8588008129579

Pole Stretchers Instructions

[Product Name] Medical Device S-2108AF and S-2108 A Pole Stretchers

[Intended Use]

The Models S-2108 AF and S-2108 A Pole Stretchers are emergency patient-handling devices. Each stretcher in the series is for professional use by a minimum of two trained operators. The stretcher can be carried by the operators.

In this manual, the generic term “stretcher” is used when the feature is shared by all models in the series. The individual stretcher name is used when features differ.

[Application]

The Models S-2108 AF and S-2108 A stretchers are the emergency stretchers of choice for: Military Medics, Athletic Trainers, First Responders, EMS, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians and more.

[Directions for use]

Folding the stretcher / Unfolding the stretchers – in opposite way



[Cleaning and Disinfection]

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time. Ferno recommends you inspect the stretcher for damage as you disinfect it.

[Inspecting the stretcher]

Have your service's equipment maintenance personnel inspect the stretcher regularly. If inspection shows damage or excessive wear, remove the stretcher from service until repair is made.

[Parts and services]

To order Ferno parts and for professional stretcher repair, contact FERNO or your Ferno distributor. Your distributor is the only agent authorized by Ferno to manage, service and repair Ferno products.

Ferno Slovakia Contact:

Ferno Slovakia s.r.o. Bošáca 893
913 07 Slovakia
Tel: + 421 (0) 32 7708 010

[Storage]

Avoid High temperature & moisture

[Declaration of Conformity]

The Models S-2108 AF and S-2108 A Pole Stretchers mentioned in this Manual were designed, manufactured and tested to meet the specification EN 1865 standards.



Please use under the guidance of the doctor or nurse