


## Model S-240



## Model S-242



CE

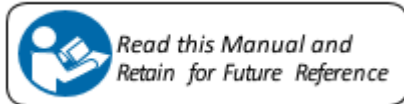

**TIP: Regular cleaning helps reduce the risk of transmitting disease and enables the equipment to function at its optimum**

## Disclaimer

This manual contains general instructions for the use, operation and care of this chair. The instructions are not all-inclusive. Safe and proper use of this chair is solely at the discretion of the user. Safety information is included as a service to the user. All other safety measures taken by the user should be within and under consideration of applicable regulations. It is recommended that training on the proper use of this chair be provided before using this chair in an actual situation. Retain this manual for future reference. Include it with the chair in the event of transfer to new users. Additional free copies are available upon request from FERNO Manufacturing.

## **WARNING**

**Untrained operators can cause injury or be injured. Permit only trained personnel to operate the chair.**

**Improper use of the chair can cause injury. Use the chair only for the purpose described in this manual.**

**Improper operation can cause injury. Operate the chair only as described in this manual.**

**An unattended patient can be injured. Stay with the patient at all times.**

**An unrestrained patient can fall off the chair and be injured. Use restraints to secure the patient on the chair.**

**Helpers can cause injury. Maintain control of the chair, operate the controls and direct all helpers.**

**Helpers can be injured. Show helpers where to grasp the chair to avoid pinch points.**

**Improper maintenance can cause injury. Maintain the chair only as described in this manual.**

**Improper parts and service can cause injury. Use only Ferno parts and Ferno-approved service on the chair.**

**Modifying the chair can cause injury and damage. Use the chair only as designed by Ferno.**

**Attaching improper items to the chair can cause injury. Use only Ferno-approved items on the chair.**

## Model S-240, S-242 Chair Product Specifications

A ..... 92 cm

B ..... 74 cm

C ..... 53 cm

D ..... 92 cm

E ..... 17 cm

F ..... 53 cm

Model S-240 ..... 8,65 kg\*

Model S-242 ..... 10 kg\*\*


\* - without restraints

\*\* - without lower rear handles

- Dimension are rounded to the nearest whole number

**Load capacity** ..... 1 Kg

**Note:** In this manual, the Model S-242 chair is used to demonstrate features common to the Model S-242 and Model S-240 chairs.

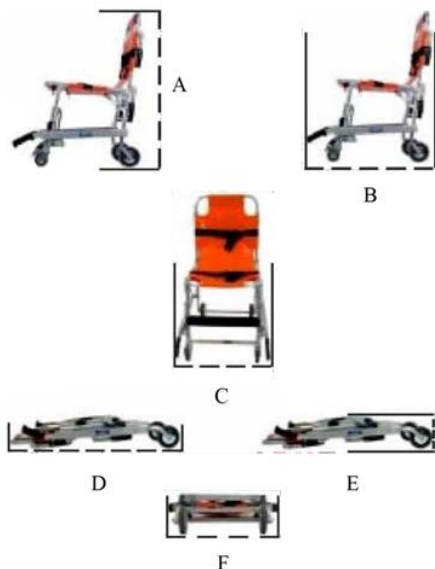

**Load Limit****Model S-240****Model S-242**

<b>Product Number</b>	<b>Product Name</b>	<b>GTIN/UDI</b>
60-0110-001	CHAIR S-240	8588008129685
60-0110-002	CHAIR S-242	8588008129685

**S-240, S-242 Chair Instructions****[Product Name] Medical Device S-240, S-242 Chair****[Intended Use]**

The Model S-240, S-242 Chair is emergency patient-handling device. Each chair in the series is for professional use by a minimum of two trained operators. The chair can be carried by the operators. In this manual, the generic term " chair " is used when the feature is shared by all models in the series. The individual chair name is used when features differ.

**[Application]**

The Model S-240, S-242 Chair is the emergency chair of choice for: Military Medics, Athletic Trainers, First Responders, EMS, Wilderness Rescue, Ski Patrols, Outdoor Adventurers, Safety Engineers, Veterinarians and more.

## [Directions for use]

Folding and unfolding the chair

### A. Unfolding the Chair



1



2



### B. Folding the Chair



1



2



3



**C. Raising Front Handles**



**D. Lowering Front Handles**



**E. Extending Front Handles**



**F. Retracting Front Handles**




**G. Back Handles**



**H Wheel Locks- Model S-242**



**I. Securing Patient in Chair with Restraints**

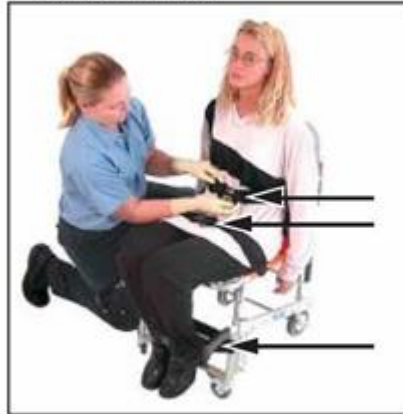

**J. Carrying Chair on**





**K. Rolling Model S-240**



**M. Rolling Model S-242**



**O. Lifting Chair Over Obstacles**



## [Cleaning and Disinfection]

Wipe all surfaces with disinfectant. Follow the disinfectant manufacturer's instructions for application method and contact time. Ferno recommends you inspect the chair for damage as you disinfect it.

## [Inspecting the chair]

Have your service's equipment maintenance personnel inspect the chair regularly. If inspection shows damage or excessive wear, remove the chair from service until repair is made.

## [Parts and services]

To order Ferno parts, and for professional chair repair, contact FERNO or your Ferno distributor. Your distributor is the only agent authorized by Ferno to manage, service and repair Ferno products.

### **Ferno Slovakia Contact:**

Ferno Slovakia s.r.o.  
Bošáca 893  
913 07 Slovakia  
Tel: + 421 (0) 32 7708 010

## [Storage]

Avoid High temperature & moisture

## [Declaration of Conformity]

The Models S-240, S-242 Chair mentioned in this Manual were designed, manufactured and tested to meet the specification of EN 1789 and EN 1865 standards.

### **Warning**

**Please use under the guidance of the doctor or nurse**

